## Instructions for Use

Universal Bio-Resorbable Synthetic Bone Graft  
Moldable Putty  
Orthopedic, Oral-Maxillofacial & Dental Surgeries  
Veterinary use only

**BIOMATCAN**  
Animal Health Division  
Unit 1, 151 Whiting Rd  
Fredericton, New Brunswick  
E3B 5Y5 Canada  
Tel: (506) 450-0404  
1-800-558-1979

info@svetoss.com  
[www.SVETOSS.us](http://www.SVETOSS.us)

### EN

#### Description:

**SVETOSS+** bioresorbable bone graft is a highly moldable, bone-like biocomposite that is fully synthetic and can be shaped or injected to fit bone defects. It is designed to maintain volume while supporting gradual replacement by the patient's own bone.

SVETOSS+ is formulated with a unique osteoconductive balanced bioceramics formulation (50% hydroxyapatite (HA) and 50% beta-tricalcium phosphate ( $\beta$ -TCP)) embedded in a biocompatible and bioresorbable pharmaceutical gel that promotes bone healing and improves the malleability, injectability, easy handling, and stability of the bone graft implant at the surgical site.

SVETOSS+ moldable putty is a fully resorbing synthetic bone graft that spares patients the trauma and risks of autograft harvesting and provides a safe alternative to animal cadaver bone. It completely eliminates the potential for immunological reactions, disease transmission, and inconsistent, unpredictable, and random results.

SVETOSS+ moldable putty is provided ready to use (RTU) in a sterile jar for single-patient usage with an ergonomic spatula for direct application. No mixing or preparation or additional tools and equipment are needed.

SVETOSS+ can be easily shaped to match the bone defect morphology and doesn't need a membrane for stabilization, which leads to decreased surgery time and costs as well as fewer complications. It also improves current treatments and widens the procedures domain. SVETOSS+ is radiopaque, safe, and has excellent biocompatibility.

#### Indications:

SVETOSS+ is an osteoconductive bone graft intended to fill, augment, or reconstruct osseous bone voids or gaps of **non-load bearing osseous defects**. Uses of SVETOSS+ modellable putty include:

#### Oral-Maxillofacial & Dental application:

- Intrabody pockets caused by periodontal disease
- Jaw fracture prevention and socket preservation after tooth extraction

- Periodontal pockets and cyst cavities
- Endo-periodontic lesions
- Traumatic defects or intraosseous flaws related to pulpal floor or lateral root perforations, and all cases where bone grafting is needed.

#### Orthopedic application:

- Void filling, osteotomy sites, and fracture repair
- Arthrodesis
- Filling voids from cysts or tumors removals
- Filling and reconstruction of metaphyseal bone defects
- Tibial tuberosity advancement (TTA)
- Tibial Plateau Leveling Osteotomy (TPLO)
- Non-union, delayed union, and mal union
- Or anywhere a bone graft material is needed
- Prosthesis revision, and all cases where bone grafting is needed.

#### Instructions for Use:

It is recommended that all procedures be performed under aseptic/surgical conditions. Routine surgical procedures should be followed to expose the surgical site. Once exposed, eliminate all granulation or necrotic tissue at the defect site.

Irrigate the defect with sterile saline or water and evacuate the excess.

There is no special handling or mixing procedures prior to use.

Inspect all device packaging prior to use to ensure maintenance of sterility.

**Step 1-** Peel open pouch and remove the sterile jar.

**Step 2-** Remove the protective cap from the jar.

**Step 3-** Using the provided spatula, take the necessary amount of the implant and fill the bone defect. Repeat this step as needed until the bone defect is completely filled.

**Step 4-** Secure the bone graft by applying gentle pressure to the biocomposite implant to ensure proper placement into the bone gap and adequate contact with the viable bone tissue.

**Step 5-** Remove any excess material and close as per standard practice. The surgeon may use cauterization or suction (if needed) to reduce bleeding.

**Step 6-** Ensure primary closure of the soft tissues over the graft site.

SVETOSS+ could be mixed with autograft and S-VETOSS granules.

#### Contra-Indications:

As with bone grafts in general, SVETOSS+ should not be used in patients with:  
Acute or chronic infection at the surgical site.  
Any existing condition or disease that will interfere with good bone and tissue healing.

#### Warnings:

SVETOSS+ is sterilized by gamma radiation and should not be re-sterilized. It is intended for single use.

Do not use if the package is opened or damaged or if the expiration date has been exceeded.

SVETOSS+ is not intended to bear mechanical loads and must not be implanted in areas that are subjected to mechanical stresses without osteosynthesis material.

Use in areas where the graft can be adequately contained.

**Do not wash or mix the implant with fluid as it may dissolve the hydrosoluble gel.**

#### Adverse effects:

Possible complications are the same as those that apply to any surgical procedure, including but not limited to: wound infection and wound complications, such as hematoma.

#### Storage:

SVETOSS+ should be stored in the original packaging at ambient temperature and in a clean environment.

#### Symbols:

	Consult Instructions for Use
	Caution, Consult accompanying Documents
	Fragile, Handle with Care
	Do Not ReUse
	Do Not ReSterilize
	Do Not Use if Package Damaged
	Expiration Date
	Batch Code
	Date of Manufacturer
	Manufacturer
	Method of Sterilization using radiation

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