

# MAINTENANCE CHECKLIST

## ⇒ DAILY

- ☐ touch up (sharpen) hand instruments
- ☐ oil handpiece

## ⇒ WEEKLY

- ☐ check compressor gauges
- ☐ drain oil free compressor
- ☐ clean handpiece turbine

## ⇒ MONTHLY

- ☐ check oil level (not below half) for oil compressors

## ⇒ YEARLY


- ☐ compressor maintenance  
filters, change oil (if have oil)
- ☐ change sterile straw for biofilm removal



*Oil the small 'large' hole*



Tip extends beyond blue line  
**Optimum Efficiency**  
(Scale Away)

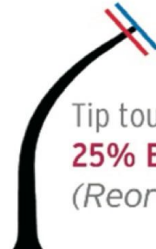

Tip touches blue line  
**25% Efficiency Loss**  
(Reorder)



Tip touches red line  
**50% Efficiency Loss**  
(Discard)

Measure and mark your new tip on this card. (ie each mark should be 1mm in real life)  
With 1mm loss, it is taking 50% longer to clean as with a brand new tip.

With a 2mm loss, it is taking **TWICE** as long to clean, so the tip should be discarded and replaced.

Please remember to ultrasonically clean with light paint brush strokes using the sides of the scaler, **NOT** the tip.

**Dr. Jennifer Mathis, DVM, DAVDC**

Board Certified Veterinary Dentist™ and Oral Surgeon  
vetceyoulluse.com  
www.tooth.vet  
ce@tooth.vet



**ANIMAL DENTISTRY**  
**REFERRAL SERVICES**

1326 Sunset Drive Norwalk, Iowa 50211